

ULTRASOUND CAVITATION SLIMMING MACHINE



● WRINKLE REMOVER

● WEIGHT LOSS

● FACE LIFTING

● SKIN TIGHTENING

● BODY RESHAPING



ONE MACHINE MULTI-FUNCTIONS

1

**Tripolar Facial
RF Handle**



2

**Multipolar Body
RF Handle**



3

**Vacuum Massage
Handle**



4

**40k Cavitation
Handle**



Function Introduction

How does Cavitation works?

The Ultrasonic Cavitation machine uses sound waves/frequencies to disrupt the fat cell walls, which causes the fat cells to "leak" their contents into the fluid spaces of your body. From there, your lymph system picks up this waste material (the loose fat) and begins circulating it through your body until it can be processed by the liver and eliminated with sweat, urine and feces.

How does RF works?

The multi-polar radio frequency causes a hot reaction in the tissue which stimulates the body's natural healing response causing new collagen to form, and the production of new elastin fibres making skin to look and feel firmer. Skin is heated consistently and uniformly without the risk of any burns.

How does Vacuum works?

After break up the subcutaneous fat, decrease cellulite accumulation. It helps smooth lymph and discharges the fatty acid and toxin that is decomposed through lymph system. Vacuum heads immediate effect in body shaping.

APPLICATIONS

- (1) Smooth fine wrinkles, shrink pores.
- (2) Make the skin moist.
- (3) Enhance lymphatic and blood circulation.
- (4) Get rid of facial redness.
- (5) Eliminate slow acne scars.
- (6) Promote collagen and cell activation.
- (7) Improve skin activity and toughness.
- (8) Increase the speed of metabolism, speed up the body to excrete waste and excessive water.
- (9) Reduce stretch marks.
- (10) Relax muscles, relieve muscle spasms, relieve muscle pain.
- (11) To tighten muscles of the arms, legs, thighs, buttocks, lower back, abdominal muscles, re-shaping body contour.
- (12) Effectively improve the orange peel-like skin of the buttocks and thighs, while also helps in postpartum or after effect of liposuction in the abdomen area.



PRODUCT DISPLAY



TREATMENT HANDPIECES



■ 40K CAVITATION HANDPIECE



■ MULTIPOLAR BODY RF HANDPIECE












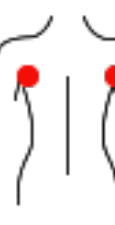


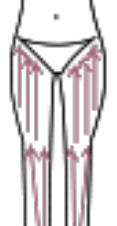



■ TRIPOLAR FACIAL RF HANDPIECE



■ VACUUM MASSAGE HANDPIECE

OPERATION GUIDANCE

<p>THIGH</p> <p>(Treated time:30min)</p>	<p>BUTTOCKS</p> <p>(Treated time:20-30min)</p>	<p>BACK</p> <p>(Treated time:20-30min)</p>	<p>ARM</p> <p>(Treated time:30min)</p>	<p>ABDOMEN</p> <p>(Treated time:20-30min)</p>
 <p>From down to upper, pushing to the groin to dredge the lymph.</p>	 <p>Along the lymph direction, pulling to the waist.</p>	 <p>First by come-and-go, pushing the bladder nerve 2 to 3 times.</p>	 <p>From the inner elbow to the armpit, doing lymph drainage.</p>	 <p>From small to large, by clockwise circling around.</p>
 <p>Circling push by anticlockwise can help decomposing fatness.</p>	 <p>From upper to down, by anticlockwise gesture, pull come-and-go.</p>	 <p>By anticlockwise circling, can help to decompose the fatness.</p>	 <p>Circling to stimulate the lymph node.</p>	 <p>From small to large, by anticlockwise direction, circling push down.</p>
 <p>Also can push by come-and-go to decompose fatness.</p>		 <p>Using lymph drainage gesture, pull the toxin to the lymph node.</p>	 <p>From the elbow to the outer, doing tightening gesture.</p>	 <p>From the belly and muscle texture, pull to the groin.</p>
 <p>Pull from the knee and muscle texture to upper, can improve the curve.</p>				 <p>Through lymph drainage gesture, taking the toxin to the place of groin.</p>

TECHNICAL PARAMETERS



Display	8 inch touch screen
Power	200W
Operating voltage	220V/50HZ; 110V/60HZ
G.W	12kg
Cavitation Frequency	40Khz
RF Frequency	1Mhz
Radio frequency power	80W
Vacuum intensity	0-100kpa

 **NUBWAY**[®] | Beauty Equipment

Beijing Nubway S & T Co., Ltd

West zone building No.5 Mauhwa Industry Park, No.1 Caida 3rd Street Shunyi District Beijing 101300 China

Website : www.nubway.net

E-mail : Info@bjnubway.com

Tel : 0086-10-60418533